
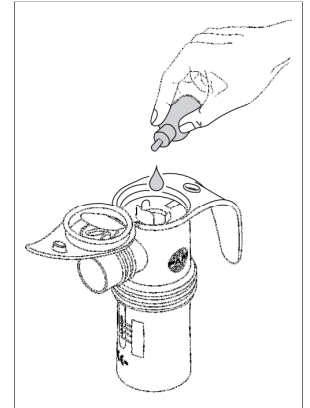


## Pari Nebulizer Kit Set-Up Procedure

### Adding the Medication


**Wash your hands.**  
Lift the hinged cap.

**Pour prescribed medication**  
Into the nebulizer cup.  
Do not fill past the  
maximum fill line

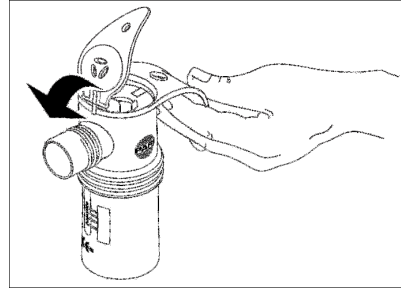

### If Using the Mouthpiece

Attach the mouthpiece to the nebulizer outlet.  
Close the hinged cap for most effective  
treatment. For faster treatment leave the  
hinged cap up.

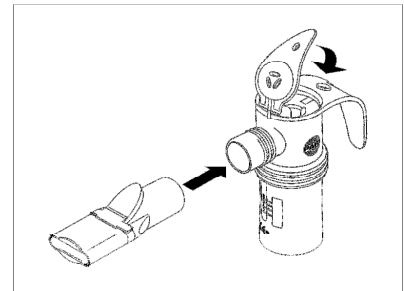

### If Using a Mask

Attach the Face Mask to the “Nebulizer Outlet”  
as shown. Close the hinged cap for most  
effective treatment. For faster treatment leave  
the hinged cap up.

